



Re-Usable Protective Mask Anti-Droplet

001- with elastic behind the ears **002-** with elastic behind the neck (produced on purpose only)

003- baby **004-** junior



PACKAGING



FRONT

BACK



PROFILE

ARTICLE: MASK /001-002-003-004

SIZE: 19x11cm ADULT UNISEX - CHILD: SIZE I (2-6 years old) 12,5x7,5cm SIZE II (6-12 years old) 13,5x8,5cm

MATERIAL: 97% COTTON 3% ELASTANE DOUBLE LAYER - 3 FOLDS – COL. WHITE IN COMPLIANCE WITH REACH STANDARDS

TYPE OF FINISHING: WATERPROOF TREATMENT WITH FLUOROCARBON RESIN (C6)

CARE INSTRUCTIONS:

MATERIA CHARACTERISTICS: WATER REPELLENT – SANITIZED – ANTI-BACTERIAL

ELASTIC: WHITE 5/6mm, POLYAMIDE 50% - ELASTOMER 50% - LATEX FREE

SEAM: POLYESTER 100%

SIZE: ONE SIZE FOR ADULTS – TWO SIZES FOR CHILDREN

USE: DISPOSABLE DEVICE FOR FACE PROTECTION – WATER- REPELLENT- FILTERING - WASHABLE- TYPE 1

Washable mask protects from spray and droplets, suitable for protection from spray and droplets which could contain dust particles, viruses or bacteria.

It is a disposable surgical mask that can be used for all environments to protect ourselves and others from possible infections transmitted via inhalation of droplets.

Wash your hands every time before and after removing the mask.

CARE:

DISINFECT THE MASK BEFORE PUTTING IT ON AND AFTER TAKING IT OFF:

WHITE MASKS: Dilute 20ml bleach (equivalent to 3 tablespoons) or Napisan detergent or Purell in one litre of water not warmer than 30° C. Leave the mask to soak for 15 minutes, do not rub, then rinse abundantly with water. Wash your hands every time before removing the mask. It is recommended to replace it after 20 washing.

DYED/ PRINTED MASKS: Dilute 20ml Napisan detergent or antibacterial suitable for colorants in one litre of water not warmer than 30°C. Leave the mask to soak for 30 minutes, do not rub, then rinse abundantly with water. Wash your hands every time before removing the mask. It is recommended to replace it after 20 washing.

EXPIRY DATE: 03/2023

STORAGE

Before storing the medical device please ensure it is clean.

- Store the medical device in cool and dry environment, protect from moisture, steam and weathering.
- Store the medical device away from heat, open flame and direct radiation from the sun.
- After usage, place the washed and completely dried mask in a box keeping it away from direct sun radiation, weathering, heat, dust, liquids and moisture.

DISPOSAL

Once the medical device has been used, please proceed with its disposal. .


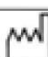




The materials used to assemble these medical devices do not need specific disposal procedures. It is necessary to follow local rules related to unsorted waste.

Avoid release to the environment.

PACKAGING: EACH DEVICE COMES WITH INSTRUCTIONS AND IN A SINGLE PACKAGE



PRODUCTION: MADE IN ITALY TAKISTON s.r.l. **DISTRIBUTED FROM:** SERIDE s.r.l.

 Produced by: Takiston S.r.l. Via Provinciale Sud, 23 – 40050 – Castello D'Argile (BO)	
Device: Washable mask	Product code: Mask 001
LOT Lot Code: LTKS921420	 Manufacturing date: 04/2020
<p>Class I medical device Class I</p> <p>The correct use and technical data are available in the maintenance manual. This mask avoids contamination to and from the patient and protects the respiratory tract from dust, spray, and other material that is used during a medical activity. Keep the medical device away from heat, open flames, direct sunlight and atmospheric agents. Store the device in a cool area.</p>    <p>Composition: Cotton 97%, Elastane 3%</p>	
	
Made in Italy	

SERIDE SRL VIA GORIZIA 13 22070 LUISAGO Fraz.PORTICHETTO (Co) TEL.031 8820111
COD.FISCALE/PARTITA IVA IT 00426570131 CAP.SOC. EURO 550.000 I.V.
N°R.E.A. 147726 – N° Registro Imprese di Como 00426570131

